



INTENDED USE.

This product should be prescribed, formed and fitted by the treating healthcare provider (HCP). Follow your HCP's instructions, including use and wearing schedule.



PRECAUTIONS.

If your **HOOK SPLINT** increases pain or worsens your condition, immediately inform your HCP. Check your skin integrity daily. If there is any unusual swelling, skin discoloration or discomfort, notify your HCP. Readjustment or remolding may be required. Do not tighten or change the configuration of your **HOOK SPLINT** unless instructed to by your provider. Dry the **HOOK SPLINT** and hand after it is submerged in water to avoid skin maceration.



CONTRAINDICATIONS.

HOOK SPLINT should not be worn over pressure sores, open wounds or irritated skin. Do not wear if the band or tabs are deformed or compromised. If these situations arise, inform your HCP.



CLEANING INSTRUCTIONS.

Clean your **HOOK SPLINT** daily. You may need to clean it more frequently depending on soil level with activity. Hand wash your **HOOK SPLINT** with warm water and non-abrasive soap. Wash and dry it gently to avoid changing its shape.



SAFEKEEPING.

When not in use, store your **HOOK SPLINT** in the zip loc bag that is provided. Clean orthosis before storing.

See **HOOKSPLINT.COM** for other languages.